

TECHNICAL MANUAL

INSTRUCTIONS FOR PROCUREMENT, ISSUE, USE AND MAINTENANCE OF MEDICAL KITS

PREPARED BY AFSC COMMODITY TEAM

THIS MANUAL SUPERSEDES TO 00-35A-39, DATED 15 AUGUST 2015.
THIS MANUAL WAS TRANSFERRED TO A NEW EDITING SYSTEM. THE STYLE AND FORMAT WAS EXTENSIVELY CHANGED/REVISED, THEREFORE, CHANGE BARS ARE OMITTED IN THIS RELEASE.

DISTRIBUTION STATEMENT A - Approved for public release; distribution is unlimited. PA Case Number AFMSA 05-001. Submit recommended changes or problems with this Technical Order to AFLCMC/EZGTP.

Published under authority of the Secretary of the Air Force

1 MARCH 2017

LIST OF EFFECTIVE PAGES

INSERT LATEST CHANGED PAGES. DESTROY SUPERSEDED PAGES.

NOTE The portion of the text affected by the changes is indicated by a vertical line in the outer margins of the page. Changes to illustrations are indicated by shaded or screened areas, or by miniature pointing hands.

Dates of issue for original and changed pages are:
 Original.0 1 March 2017

TOTAL NUMBER OF PAGES IN THIS PUBLICATION IS 42, CONSISTING OF THE FOLLOWING:

Page No.	*Change No.	Page No.	*Change No.	Page No.	*Change No.
Title	0				
A	0				
i - v	0				
vi Blank	0				
1-1 - 1-4	0				
2-1 - 2-7	0				
2-8 Blank	0				
3-1 - 3-21	0				
3-22 Blank	0				

* Zero in this column indicates an original page.

TABLE OF CONTENTS

Chapter	Page
LIST OF ILLUSTRATIONS	iv
LIST OF TABLES	iv
INTRODUCTION	v
1 INSTRUCTIONS FOR PROCUREMENT, ISSUE, USE AND MAINTENANCE OF MEDICAL KITS	1-1
1.1 Purpose	1-1
1.2 Authorization for Use and Issue Procedures	1-1
1.2.1 Authorizations	1-1
1.2.2 Initial Issue	1-1
1.2.3 Usage	1-1
1.2.4 Substitutes	1-1
1.3 Serviceability Tags	1-2
1.3.1 Markings	1-2
1.3.2 Purpose	1-2
1.4 Inspection of Medical Kits	1-2
1.4.1 Inspection by MLF	1-2
1.4.2 Inspection Procedures	1-3
1.5 Inspection By Other Than Medical Personnel	1-3
1.5.1 Inspection by Using Activity	1-3
1.5.2 Inspection by Non-medical Units	1-3
1.6 Funding for Initial/Replacement Kits	1-3
1.6.1 Initial Issue Funding	1-3
1.6.2 Mass Casualty Funding	1-3
1.6.3 Replacement Funding	1-3
1.7 Turn In of Excess or unserviceable Kits	1-3
1.7.1 Disposition of Excess	1-3
1.7.2 Disposition of Aircraft	1-4
1.7.3 Disposition Guidance	1-4
1.8 Kit Standardization or Modification Procedures	1-4
2 FIRST AID KIT MODULES	2-1
2.1 NSN 6545-01-525-9847, Trauma Module	2-1
2.1.1 Purpose	2-1
2.1.2 Installation	2-1
2.1.3 Marking	2-1
2.1.4 Inspection	2-1
2.1.5 Sealing/Packing	2-1
2.1.6 Sustainment	2-1
2.2 NSN 6545-01-525-9849 Minor Module	2-2
2.2.1 Purpose	2-2
2.2.2 Installation	2-2
2.2.3 Marking	2-2
2.2.4 Inspection	2-2
2.2.5 Sealing/Packing	2-2
2.2.6 Sustainment	2-2
2.3 NSN 6545-01-526-0062, Splint Module	2-3
2.3.1 Purpose	2-3
2.3.2 Installation	2-3

TABLE OF CONTENTS - CONTINUED

Chapter		Page
2.3.3	Marking	2-3
2.3.4	Inspection	2-3
2.3.5	Sealing/Packing	2-3
2.3.6	Sustainment	2-3
2.4	NSN 6545-01-526-0065, CPR Module	2-4
2.4.1	Purpose	2-4
2.4.2	Installation	2-4
2.4.3	Marking	2-4
2.4.4	Inspection	2-4
2.4.5	Sealing/Packing	2-4
2.4.6	Sustainment	2-4
2.5	NSN 6545-01-534-0935, Survival Module	2-5
2.5.1	Purpose	2-5
2.5.2	Installation	2-5
2.5.3	Marking	2-5
2.5.4	Inspection	2-5
2.5.5	Sustainment	2-5
2.6	NSN 6545-01-534-0925, Medical Module	2-6
2.6.1	Purpose	2-6
2.6.2	Installation	2-6
2.6.3	Marking	2-6
2.6.4	Inspection	2-6
2.6.5	Sustainment	2-7
3	FIRST AID KIT	3-1
3.1	NSN 6545-01-632-0167, First Aid Kit, Joint (JFAK)	3-1
3.1.1	Purpose	3-1
3.1.2	Marking	3-1
3.1.3	Inspection	3-1
3.1.4	Sealing/Tagging	3-1
3.1.5	Packing	3-1
3.1.6	Sustainment	3-1
3.2	NSN 6545-01-528-6546, First Aid Kit Individual (IFAK)	3-4
3.2.1	Installation	3-4
3.2.2	Marking	3-4
3.2.3	Inspection	3-4
3.3	NSN 6545-01-644-0653, First Aid Kit, AFSOC	3-5
3.3.1	Purpose	3-5
3.3.2	Installation	3-5
3.3.3	Marking	3-5
3.3.4	Inspection	3-5
3.3.5	Sealing/Tagging	3-5
3.3.6	Packing	3-5
3.3.7	Sustainment	3-5
3.4	NSN 6545-01-533-7043, First Aid Kit, Aircraft Panel Mounted	3-7
3.4.1	Installation	3-7
3.4.2	Marking	3-7
3.4.3	Inspection	3-7
3.4.4	Packing	3-7
3.5	NSN 6545-01-659-8286, FIRST AID KIT, AIRCRAFT PANEL MOUNTED JFAK	3-9
3.5.1	Purpose	3-9
3.5.2	Installation	3-9
3.5.3	Markings	3-9
3.5.4	Inspection	3-9

TABLE OF CONTENTS - CONTINUED

Chapter		Page
3.5.5	Sealing/Tagging	3-9
3.5.6	Packing	3-9
3.5.7	Sustainment.	3-9
3.6	NSN 6545-01-534-0779, First Aid Kit, General Purpose.	3-11
3.6.1	Purpose	3-12
3.6.2	Installation	3-12
3.6.3	Marking	3-12
3.6.4	Inspection	3-12
3.6.5	Sealing/Tagging	3-12
3.6.6	Packing	3-12
3.6.7	Sustainment.	3-12
3.7	NSN 6545-01-534-0894, Survival Kit, Escape And Evasion (E & E)	3-13
3.7.1	Purpose	3-13
3.7.2	Installation	3-13
3.7.3	Marking	3-13
3.7.4	Inspection	3-13
3.7.5	Sustainment.	3-13
3.8	6545-01-526-0423, First Aid Kit, Mass Casualty	3-14
3.8.1	Purpose	3-14
3.8.2	Marking	3-14
3.8.3	Inspection	3-14
3.8.4	Sealing/Tagging	3-14
3.8.5	Packing	3-15
3.8.6	Sustainment.	3-15
3.9	NSN 6910-01-528-9455, Self-Aid Buddy Care (SABC) Kit	3-18
3.9.1	Purpose	3-18
3.9.2	Marking	3-18
3.9.3	Inspection	3-18
3.9.4	Sealing/Packing	3-18
3.9.5	Sustainment.	3-18

LIST OF ILLUSTRATIONS

Number	Title	Page
1-1	AFTO Form 104 and DD Form 1574	1-2
2-1	NSN 6545-01-525-9847, Trauma Module	2-2
2-2	NSN 6545-01-525-9849, Minor Module	2-3
2-3	NSN 6545-01-526-0062, Splint Module	2-4
2-4	NSN 6545-01-526-0065, CPR Module	2-5
2-5	NSN 6545-01-534-0935, Survival Module	2-6
2-6	NSN 6545-01-534-0925, Medical Module	2-7
3-1	NSN 6545-01-632-0167, First Aid Kit, Joint (JFAK)	3-3
3-2	NSN 6545-01-528-6546, First Aid Kit, Individual (IFAK).	3-4
3-3	NSN 6545-01-644-0653, First Aid Kit, AFSOC	3-6
3-4	NSN 6545-01-533-7043, First Aid Kit, Aircraft Panel Mounted	3-8
3-5	NSN 6545-01-629-8286, First Aid Kit, Aircraft Panel Mounted JFAL.	3-10
3-6	CPR Module	3-11
3-7	Aircraft Minor Module	3-11
3-8	Aircraft Trauma Module	3-11
3-9	NSN 6545-01-534-0779, First Aid Kit, General Purpose	3-13
3-10	NSN 6545-01-537-0894, Survival Kit, Escape and Evasion (E & E).	3-14
3-11	NSN 6545-01-526-0423, First Aid Kit, Mass Casualty (Front).	3-15
3-12	NSN 6545-01-526-0423, First Aid Kit, Mass Casualty (Back)	3-16
3-13	NSN 6545-01-526-0423, First Aid Kit, Mass Casualty (Unpacked).	3-17
3-14	NSN 6545-01-526-0423, First Aid Kit, Mass Casualty (Packed).	3-18
3-15	NSN 6910-01-528-9455, Self-Aid Buddy Care (SABC) Instructor Kit	3-19
3-16	NSN 6910-01-528-9455, Self-Aid Buddy Care (SABC) Instructor Kit (Packed)	3-20
3-17	NSN 6910-01-528-9455, Self-Aid Buddy Care (SABC) Instructor Kit (Unpacked)	3-21

LIST OF TABLES

Number	Title	Page
1-1	First Aid Kit Reference Chart	1-1
2-1	NSN 6545-01-525-9847, Trauma Module - Components	2-1
2-2	NSN 6545-01-525-9849, Minor Module - Components	2-2
2-3	NSN 6545-01-526-0062, Splint Module - Components	2-4
2-4	NSN 6545-01-526-0065, CPR Module - Components	2-5
2-5	NSN 6545-01-534-0935, Survival Module	2-5
2-6	NSN 6545-01-534-0925, Medical Module	2-7
3-1	NSN 6545-01-632-0167 First Aid Kit, Joint - Components	3-1
3-2	NSN 6545-01-528-6546, First Aid Kit, Individual (IFAK) - Components	3-4
3-3	NSN 6545-01-644-0653, First Aid Kit, AFSOC - Components	3-5
3-4	NSN 6545-01-533-7043, First Aid Kit, Aircraft Panel Mounted - Components	3-7
3-5	NSN 6545-01-629-8286, First Aid Kit, Aircraft Panel Mounted JFAK - Components	3-9
3-6	NSN 6545-01-534-0779, First Aid Kit, General Purpose - Components	3-12
3-7	NSN 6545-01-534-0894, Survival Kit, Escape and Evasion (E & E) - Components	3-13
3-8	NSN 6530-01-432-5114, LITTER, Raven	3-15
3-9	NSN 6545-01-526-0423, First Aid Kit, Mass Casualty - Components	3-15
3-10	NSN 6910-01-528-9455, Self-Aid Buddy Care Instructor Kit (SABC) - Components	3-18
3-11	Supplemental JFAK SABC Components - P/N RC-0076.	3-19

INTRODUCTION

1 PURPOSE.

The objective of this technical manual is to list the purpose of medical first aid or survival kits, modules and/or components authorized for use by USAF activities. Describe the procedures to standardize or modify medical first aid or survival kits. As well as, delineate responsibilities for authorizing, requisitioning, issuing, installing, inspecting, controlling and disposing of medical kits.

2 ABBREVIATIONS AND ACRONYMS.

For definitions of standard abbreviations used throughout this manual, refer to ASME Y14.38M. Non standard abbreviations used in this manual are defined below:

AFAK	Aircraft Panel Mounted First Aid Kit
AFMOA	Air Force Medical Operations Agency
AFOSH	Air Force Occupational Safety and Health
AFPDO	Air Force Publishing Distribution Office
AFSOC	Air Force Special Operations Command
AFTO	Air Force Technical Order
AFI	Air Force Instruction
AS	Allowance Standard
CAGE	Commercial and Government Entity
DOE	Date of Expiration
DOM	Date of Manufacturer
IAW	In Accordance With
IFAK	Individual First Aid Kit
JFAK	Joint First Aid Kit
MAJCOM	Major Command
MLF	Medical Logistics Flight
MOU	Memorandum of Understanding
MTF	Military Treatment Facility
NSN	National Stock Number
SABC	Self-Aid Buddy Care
SAFK	Shelter First Aid Kit
TO	Technical Order
TQ	Tourniquet
WRM	War Reserve Materiel

3 IMPROVEMENT REPORTS.

Recommendations for improvements to prescribed requirements and procedures will be submitted by AFTO Form 22, Technical Order System Publication Improvement Report, in accordance with TO 00-5-1. Completed forms will be forwarded to AFLCMC/LZPTP, Tinker Air Force Base, OK 73145.

CHAPTER 1

INSTRUCTIONS FOR PROCUREMENT, ISSUE, USE AND MAINTENANCE OF MEDICAL KITS

1.1 PURPOSE.

The objective of this technical manual is to list the purpose of medical first aid or survival kits, modules and/or components authorized for use by USAF activities. Describe the procedures to standardize or modify medical first aid or survival kits. As well as, delineate responsibilities for authorizing, requisitioning, issuing, installing, inspecting, controlling and disposing of medical kits.

1.2 AUTHORIZATION FOR USE AND ISSUE PROCEDURES.

1.2.1 Authorizations. Authorizations may be granted by the Military Treatment Facility (MTF) Commander IAW AFI 41-209, *Medical Logistics Support*, Chapter 3. Organizations requiring issue of medical kits not specifically authorized by a current Allowance Standard (AS) will cite the appropriate Technical Order (TO) as the authority for issue.

1.2.2 Initial Issue. Items listed in this TO are initially issued to using activities on a reimbursable basis by the local Medical Logistics Flight (MLF). Initial issue requests are submitted to the MLF for verification of TO or AS authorization. Without authorization, requests are submitted through the MLF to the MTF Commander for review and approval. Requests for replacement kits are forwarded directly to the MLF. AF units not supported by an Air Force MTF will order initial issue and replacement kits through the supporting Service MTF. Sustainment support for the assigned kits will be coordinated and established through AFMOA/SGALX.

1.2.3 Usage. The kits listed are designed for a specific purpose/use and will not be modified, disassembled, issued, or used in any manner not prescribed herein, including training and exercises. The kits need to be ready and completely assembled for their intended purpose.

1.2.4 Substitues. When required to keep flight crews mission ready, the MTF Commander may approve temporary substitutes after consulting with the senior Wing flight surgeon. Temporary substitution kits will be marked with an expiration date of 30 days. Permanent substitutes will be coordinated with AFMOA/SGALX. Medical kits will not be used when other medical services and supplies are readily available.

Table 1-1. First Aid Kit Reference Chart

Authorized First Aid Kit NSNs	First Aid Kit Item Description
6545-01-632-0167	First Aid Kit, Joint (JFAK)
6545-01-528-6546	First Aid Kit, Individual (IFAK)
6545-01-644-0653	First Aid Kit, Individual AFSOC NOTE: For AFSOC Personnel Use Only!
6545-01-534-0779	First Aid Kit, General Purpose
6545-01-533-7043	First Aid Kit, Aircraft Panel Mounted
6545-01-659-8286	First Aid Kit, Aircraft Panel Mounted JFAK
6545-01-534-0894	First Aid Kit, Survival (E&E)
6545-01-526-0423	First Aid Kit, Mass Casualty
6910-01-526-9455	Self-Aid Buddy Care Instructor Training Kit (SABC)
<ul style="list-style-type: none"> • Kits contain shelf life dated materiel requiring replacement and management; refer to specific kits for details. • Note code “ P” indicates shelf life materiel. 	

1.3 SERVICEABILITY TAGS.

1.3.1 **Markings.** All first aid kits will be tagged with a DD 1574 to certify NSN, expiration date, lot number, and serviceability. If the DD Form 1574 becomes detached and a new 1574 is unavailable, an AFTO Form 104 may be used.

1.3.2 **Purpose.** The tag or label will be marked as shown in Figure 1-1. Major commands are authorized to grant aircrew flight equipment activities the option to delete or obliterate the inspecting activity when aircrew flight equipment requires sanitizing. If the manufacturer has not attached a DD Form 1574 on new kits, the MLF will attach a completed tag on initial issue to indicate that the components will not expire prior to the next inspection date. In addition to inspection information, tags protect the integrity of medical kits. Kits with a torn AFTO Form 104 or missing DD Form 1574 are not issued. These kits are returned to the MLF for re-inspection.

NOTE

- The AFTO Form 104 must be ordered through AFPDO <https://wmsweb.afncr.af.mil/wms/> (select E-mail certificate when logging in with CAC). Keyword Search: AFTO 104.
- Escape and Evasion medical **WILL NOT** be tagged with a DD Form 1574 or an AFTO Form 104; ensure serviceability of manufacturer’s marking/labeling. Using activities requiring replacement modules will provide funding to/order modules through the MLF. Re-inspections are coordinated through your MLF.

WARNING: Unauthorized persons removing, defacing or destroying this tag may be subject to a fine of not more than \$1,000 or imprisonment for not more than one year or both. (18 USC 1361)	FSN, PART NO. AND ITEM DESCRIPTION 6545-01-632-0167 First Aid Kit, Joint		SERVICEABLE TAG-MATERIAL	
			NEXT INSPECTION DUE/ OVERAGE DATE Feb. 2020	CONDITION CODE
			INSPECTION ACTIVITY FM4425	
	SERIAL NUMBER/LOT NO.	UNIT OF ISSUE EA	INSPECTOR'S NAME OR STAMP AND DATE Inspector Gadget	
	CONTRACT OR PURCHASE ORDER NO.	QTY 1	Feb. 2015	
REMARKS TO 00-35A-39				

Previous edition may be used
 DD FORM 1574, OCT 88
 SN/ 0102-LF-014-5600

FIRST AID/SURVIVAL KIT INSPECTION CERTIFICATE		
NEXT INSPECTION DUE/ OVERAGE DATE: 6 May 2017		
INSPECTION ACTIVITY Andrews AFB Md, 20762	INSPECTOR FM4425	DATE 6 May 2014
AFTO FORM APR 71 104		

H9003172

Figure 1-1. AFTO Form 104 and DD Form 1574

1.4 INSPECTION OF MEDICAL KITS.

1.4.1 **Inspection by MLF.** Kits listed in this TO will be inspected by the MLF IAW their expiration cycle/ the majority are at 36 months shelf life. Off-cycle inspections may occur when suspension of the kit or one of its components/modules is announced via Air Force Medical Logistics or Medical Material Quality Control message, or when turned in by using

activities IAW Paragraph 1.7. Medical logistics personnel will ensure sufficient shelf life is available on kit components/modules under this program. Using activities will ensure kits are removed from use and turned in to the MLF for inspection, quality assurance and/or reconstitution action prior to the next inspection due and/or overage date or immediately upon receiving notice from the MLF of an applicable item suspension.

1.4.2 Inspection Procedures. During the inspection, the MLF will accomplish the following minimum actions.

1.4.2.1 Properly mark and tag kits IAW Paragraph 1.3 and Paragraph 1.5.

1.4.2.2 Replace all expiration dated items/modules with less than 6 months shelf life remaining. Otherwise, the NEXT INSPECTION DUE AND/OR OVERAGE DATE annotated on the serviceability tag will be that of the component/module that has the shortest expiration date or shelf life remaining.

1.4.2.3 If applicable, components/modules will be replaced by items with a later date of manufacture during normal inspection of the kits.

1.4.2.4 Ensure that DLAR-4155-37, Materiel Quality Control Storage Standards (Appendix M), is used as a reference in the inspection of all kits and components.

1.4.2.5 Component items/modules of first aid and/or survival kits that are deleted through normal supply action will be replaced or withdrawn from the sets and kits at the next normally scheduled inspection cycle. It is not necessary to recall the kits in use to effect replacement of deleted items unless otherwise notified through appropriate channels.

1.4.2.6 If replacement of the first aid kit or its content is required, then it will be charged to the applicable USAF activity.

1.4.2.7 It is not necessary to recall the kits in use to effect the changes. This is in effect for two years from the date of technical order publication.

1.5 INSPECTION BY OTHER THAN MEDICAL PERSONNEL.

1.5.1 Inspection by Using Activity. Using activities are responsible for periodic inspection to ensure serviceability of kits prior to issue or while in use. Be mindful of environmental exposure. Opened kits or those whose outer appearance indicates possible damage will be turned in to the MLF for inspection, quality control and re-tagging/markings. Inspections by using activities are limited to external examination of kit external container, seals and tag/markings. To prevent large numbers of kits being removed from service at-a-time, plan inspections and purchases with current lead times. The DD Form 1574 or AFTO Form 104 may only be removed by the MLF.

1.5.2 Inspection by Non-medical Units. Non-medical units, such as an Expeditionary Theater Distribution Center, may perform these TO requirements if the theater Surgeon and A4 agree and have established a Memorandum of Understanding (MOU).

1.6 FUNDING FOR INITIAL/REPLACEMENT KITS.

1.6.1 Initial Issue Funding. Initial issue of kits listed in Table 1-1 is funded by requesting/requiring units or agencies, if other than Air Force. Refer to Paragraph 1.2.2 for additional issue guidance.

1.6.2 Mass Casualty Funding. Mass Casualty Kits supporting WRM assemblies are funded by AFMOA/SGALX.

1.6.3 Replacement Funding. Replacement of kit components/modules, e.g., exterior case/pouch, interior components, etc. is funded by the requesting/requiring units or agencies. Unserviceable/expired kit components as listed in Chapter 2 and modules as listed in Chapter 3 are replaced one-for-one.

1.7 TURN IN OF EXCESS OR UNSERVICEABLE KITS.

1.7.1 Disposition of Excess. Turn-in excess components generated during inspections to the MLF for processing through the stock record account in accordance with AFI 41-209, Chapter 3.

TO 00-35A-39

1.7.2 Disposition of Aircraft. Medical first aid kits will be removed from aircraft prior to disposal or transfer of the aircraft to the Military Aircraft Storage and Disposition Center. Using activities are responsible for returning these and all other excess kits to the MLF.

1.7.3 Disposition Guidance. Unserviceable kits/modules will be turned into the MLF for disposal. Kits/modules will be disposed of through normal channels, (i.e. DRMO or in-house destruction). Kits/module destructions performed in house must be accomplished IAW AFI 41-209, Paragraph 3.17.6.

1.8 KIT STANDARDIZATION OR MODIFICATION PROCEDURES

Recommendations to standardize or modify additional medical kits will be submitted using AFTO Form 252, Technical Order Publication Change Request. Recommendations will be coordinated through both medical and aircrew flight equipment authorities at each level of command.

CHAPTER 2

FIRST AID KIT MODULES

2.1 NSN 6545-01-525-9847, TRAUMA MODULE.

Refer to Figure 2-1

2.1.1 Purpose. The Trauma Module provides Airmen first aid capability to treat injuries sustained in the line-of-duty.

2.1.2 Installation. This module contains components of the AFAK, General Purpose First Aid Kit, IFAK/AFSOC, and Mass Casualty Kit.

2.1.3 Marking. Markings includes the NSN, module name, Commercial and Government Entity (CAGE) code, expiration date of component with shortest shelf life, inspection/test date and lot (control) number, and a list of module components.

2.1.4 Inspection. Refer to Paragraph 1.4 and Paragraph 1.5 for inspection guidance.

2.1.5 Sealing/Packing. This module is in a waterproof/vacuum sealed enclosure. If the integrity of the vacuum seal is opened or otherwise compromised, this item must be returned to the MLF for replacement.

2.1.6 Sustainment. Trauma module has a 36 month non extendable shelf-life. Replacement modules must be ordered 6 months prior to expiration to ensure units remain serviceable. Refer to Paragraph 1.2.2 and Paragraph 1.6.3 for replacement guidance.

Table 2-1. NSN 6545-01-525-9847, Trauma Module - Components

Notes	Description	Qty/UI
P	BANDAGE KIT, Elastic	2 EA
P	BANDAGE GAUZE, Cotton, 4.5 in x 4.1 yd	2 EA
P	ADHESIVE TAPE, Surgical, 1 in x 10 yd	1 EA
	AIRWAY, Nasopharyngeal	1 EA
	SHEARS, EMS, (5-1/2 in, Black)	1 EA



H0901307

Figure 2-1. NSN 6545-01-525-9847, Trauma Module

2.2 NSN 6545-01-525-9849 MINOR MODULE .

Refer to Figure 2-2.

2.2.1 **Purpose.** The Minor Module provides Airmen first aid capabilities to treat injuries sustained in the line-of-duty.

2.2.2 **Installation.** Component of the AFSOC FAK, AFAK, General Purpose FAK, IFAK and Mass Casualty Kit.

2.2.3 **Marking.** Markings include the NSN, module name, Commercial and Government Entity (CAGE) code, expiration date component with shortest shelf life, inspection/test date and lot (control) number, and a list of module components.

2.2.4 **Inspection.** Refer to Paragraph 1.4 and Paragraph 1.5 for inspection guidance.

2.2.5 **Sealing/Packing.** This module is in a waterproof/vacuum sealed enclosure. If the integrity of the vacuum seal is opened or otherwise compromised, this item must be returned to the MLF for replacement.

2.2.6 **Sustainment.** Minor module has a 36 month non extendable shelf-life. Replacement modules must be ordered six months prior to expiration to ensure units remain serviceable. Refer to Paragraph 1.2.2 and Paragraph 1.6.3 for replacement guidance.

Table 2-2. NSN 6545-01-525-9849, Minor Module - Components

Notes	Description	Qty/UI
P	BACITRACIN OINTMENT, USP, 0.0312 oz (0.8736 gm)	3 EA
P	REHYDRATION SALTS, Oral (Modified Lemon Flavor, 50 gm)	1 EA
P	LIPSTICK, Antichap (Cold Climate and Hot Climate, 4.2 gm)	1 EA
P	BANDAGE, Adhesive, 3/4 in x 3 in	10 EA
P	BANDAGE, Adhesive, 2 in x 4 in	5 EA

Table 2-3. NSN 6545-01-526-0062, Splint Module - Components

Description	Qty/UI
SPLINT, Universal (Grey)	2 EA
ELASTIC BANDAGE, 4 in x 2 yds	1 EA



H1501527

Figure 2-3. NSN 6545-01-526-0062, Splint Module

2.4 NSN 6545-01-526-0065, CPR MODULE .

Refer to Figure 2-4.

2.4.1 Purpose. The CPR Module provides Airmen first aid capabilities to treat injuries sustained in the line-of-duty.

2.4.2 Installation. This module is to be used in the Aircraft First Aid Kit (AFAK), Mass Casualty Kit, and the General Purpose First Aid Kit.

2.4.3 Marking. Markings include the NSN, module name, CAGE code, expiration date of the component having the shortest shelf life, inspection/test date and lot number, and a list of module components.

2.4.4 Inspection. Refer to Paragraph 1.4 and Paragraph 1.5 for inspection guidance.

2.4.5 Sealing/Packing. This module is in a waterproof/vacuum sealed enclosure. If the integrity of the vacuum seal is opened or otherwise compromised, this item must be returned to the MLF for replacement.

2.4.6 Sustainment. Refer to Paragraph 1.2.2 and Paragraph 1.6.3 for replacement guidance.

Table 2-4. NSN 6545-01-526-0065, CPR Module - Components

Description	Qty/UI
CPR MICRO SHIELD, P/N 51001M	1 EA
GLOVES, Exam (Nitrile, Sz: Large)	4 EA



H1501528

Figure 2-4. NSN 6545-01-526-0065, CPR Module

2.5 NSN 6545-01-534-0935, SURVIVAL MODULE.

Refer to Figure 2-5.

2.5.1 **Purpose.** The Survival Module provides Airmen first aid capability to treat injuries sustained in the line-of-duty.

2.5.2 **Installation.** This module is to be used in the Survival Kit. E & E Kit modules will be attached IAW TO 14S1-3-51. Grommet will be used to secure the lanyard to the E & E kit modules. Vent holes are not required to be punched in outer bag for installation into survival kit.

2.5.3 **Marking.** Markings on the outer module bag label will be in bold upper case lettering and will include the NSN, module name, CAGE code, expiration date, date of manufacture and lot number. The NSN, CAGE code, DOE, DOM, Lot # are also displayed in linear bar codes.

2.5.4 **Inspection.** Refer to Paragraph 1.4 and Paragraph 1.5 for inspection guidance.

2.5.5 **Sustainment.** Aircrew Flight Equipment units with MAJCOM authorization may replace components in the survival module to extend the service life. MAJCOM will provide instructions to aircrew flight equipment units for accomplishing this task. Refer to Paragraph 1.2.2 and Paragraph 1.6.3 for replacement guidance.

Table 2-5. NSN 6545-01-534-0935, Survival Module

Notes	Description	Qty/UI
	LEATHERMAN MINI TOOL, W/Pliers (Gray)	1 EA
	MATCH CASE, Watertight (Plastic)	1 EA

Table 2-5. NSN 6545-01-534-0935, Survival Module - Continued

Notes	Description	Qty/UI
	FLINT STRIKER, Mini Sparking (in Match Case)	1 EA
	LIFEBOAT MATCHES, Flat Pack (in Match Case)	10 EA
	STRIKER BOARDS, in Match Case	2 EA
	MINI COMPASS in Match Case	1 EA
P	WATER PURIFICATION TABLETS	10 EA
P	WETFIRE FUEL CUBES	2 EA
	LASER LIGHT FINGER POINTS (Visible Red)	1 EA
	SIGNAL MIRROR, W/Lanyard	1 EA
	FLAT WHISTLE, W/Lanyard	1 EA
	LASER LIGHT FINGER POINTER (Infra-red/Ground)	1 EA
	INFRA-RED GLOW TAPE (Velcro App, 1 in x 1 in)	2 EA
	CAMOUFLAGE CRAVAT, 37 in x 52 in (w/two 2-in #3 safety pins)	1 EA



H1501529

Figure 2-5. NSN 6545-01-534-0935, Survival Module

2.6 NSN 6545-01-534-0925, MEDICAL MODULE.

Refer to Figure 2-6.

2.6.1 Purpose. The Medical Module provides Airmen first aid capability to treat injuries sustained in the line-of-duty.

2.6.2 Installation. This module is to be used in the Survival Kit, E & E Kit.

2.6.3 Marking. Markings will be in bold upper case lettering and will include the NSN, module name, CAGE code, expiration date, date of manufacture, and lot number. The NSN, CAGE code, DOE, DOM, and Lot # are also displayed in linear bar codes.

2.6.4 Inspection. Refer to Paragraph 1.4 and Paragraph 1.5 for inspection guidance.

2.6.5 Sustainment. Medical module has a 36 month non extendable shelf-life. Replacement modules must be ordered six months prior to expiration to ensure units remain serviceable. Refer to Paragraph 1.2.2 and Paragraph 1.6.3 for replacement guidance.

Table 2-6. NSN 6545-01-534-0925, Medical Module

Notes	Description	Qty/UI
P	ADHESIVE BANDAGE, 1 in x 3 in	4 EA
P	ADHESIVE BANDAGE, 2 in x 4-1/2 in	4 EA
P	DRESSING, GAUZE (4-1/2 in x 4-1/10 yds)	1 EA
P	FLAT PACK CAMOUFLAGE WRAP, 2 in x 2 yds	1 EA
P	BACITRACIN OINTMENT 0.9 gm	2 EA
P	ANTI-BACTERIAL TOWELETTES	2 EA
P	SPF 30 SUN PROTECTION TOWELETTE	1 EA
P	30% DEET TOWELETTE	1 EA
P	CERALYTE 70, Lemon	1 EA
	SAFETY PINS, 2 in, #3	4 EA
P	Quik Clot Combat Gauze™ 3 in x 4 yds	1 EA
	CYALUME SOS Infra-red Chemlight (4 in, 8 hr/NSN: 6260-01-533-1547)	



H1501530

Figure 2-6. NSN 6545-01-534-0925, Medical Module

CHAPTER 3 FIRST AID KIT

3.1 NSN 6545-01-632-0167, FIRST AID KIT, JOINT (JFAK).

Refer to Figure 3-1.

3.1.1 Purpose. The JFAK provides Airmen first aid capability to treat injuries sustained in the line-of-duty. The JFAK replaces the Individual First Aid Kit 6545-01-528-6546. When issued as an item of individual equipment on AF form 538 and 1297, the kit will be worn on the vertical strap of field clothing except during bail out procedures.

3.1.2 Marking. Markings on each kit shall include the nomenclature, NSN (for the kit and case), and the expiration date of the earliest expiring component. Special Markings include a black plus sign (+) to indicate medical kit. Tourniquet (TQ) markings include “TQ” in bold black lettering, NSN, Manufacturer, and date of manufacture/lot.

3.1.3 Inspection. Using activities will return unserviceable kits to the MLF for inspection/reconstitution. Kits are considered unserviceable if the DD form 1574 is not attached or any components are expired. Inspect both TQ and pouch. Inspect buckles, rod and locking clips on TQ for chips/cracks, ensure straps are not split, frayed or over stretched. Return unserviceable TQ’s to MLF for disposal. Refer to Paragraph 1.4 and Paragraph 1.5 for further inspection guidance.

3.1.4 Sealing/Tagging. Prior to issuing this kit to using activities, the MLF will ensure the kit has been tagged as outlined in Paragraph 1.4.

3.1.5 Packing. Components are packaged individually. If the integrity of the packaging is compromised, the component is considered unserviceable and must be returned to the MLF for disposal.

3.1.6 Sustainment. The JFAK has a 60 month non extendable shelf-life. Six months prior to expiration replacements should be ordered, upon expiration the JFAK will be turned into the MLF for disposal. Refer to Paragraph 1.2.2 and Paragraph 1.6.3 for replacement guidance.

NOTE

- JFAK without Tourniquet Configuration - 6545-01-643-8543.
- Units may purchase a JFAK without tourniquets ONLY if serviceable tourniquets from IFAKs are available.

Table 3-1. NSN 6545-01-632-0167 First Aid Kit, Joint - Components

Notes	Description	Qty/UI
	BAG, USAF JFAK, (Tiger Stripe)	1 EA
	SAFETY HOOK	1 EA
	MARKER, Mini (Sharpie®)	1 EA
	EYE SHIELD, Rigid	2 EA
P	AIRWAY, Nasopharyngeal	1 EA
P	CHEST SEAL, Vented (HyFin®)	1 EA
P	PRESSURE BANDAGE	2 EA
P	COMPRESSED GAUZE	2 EA
P	GLOVES, Paper Bound	2 EA
P	CRAVAT	1 EA
P	LUBRICATING JELLY	1 EA
P	COMBAT MEDIC TAPE, Roll	1 RL
P	QuikClot COMBAT GAUZE™	2 EA
	TOURNIQUET, Combat Application/C-A-T®, (One-Handed)	2 EA

Table 3-1. NSN 6545-01-632-0167 First Aid Kit, Joint - Components - Continued

Notes	Description	Qty/UI
	FIELD MEDICAL CARD, DD Form 1380	1 EA



H9003177

Figure 3-1. NSN 6545-01-632-0167, First Aid Kit, Joint (JFAK)

3.2 NSN 6545-01-528-6546, FIRST AID KIT INDIVIDUAL (IFAK).

Refer to Figure 3-2. The IFAK has been replaced by the JFAK 6545-01-632-0167 and is no longer authorized to order. The IFAK is however, authorized for use in contingency operations until full implementation of the JFAK.

3.2.1 **Installation.** When issued as an item of individual equipment on AF Form 538 or 1297, the kit will be worn on a vertical strap of field clothing except during bail out procedures.

3.2.2 **Marking.** Markings on each kit will include nomenclature, NSN (for the kit and case), and the expiration date of the earliest expiring module.

3.2.3 **Inspection.** Using activities will return unserviceable kits to the MLF for disposal. Kits are considered unserviceable if the DD form 1574 is not attached, any of the modules or components are expired, or the integrity of the vacuum seal is compromised

Table 3-2. NSN 6545-01-528-6546, First Aid Kit, Individual (IFAK) - Components

Notes	Description	Qty/UI
	CASE, FIRST AID KIT, UNIVERSAL PRINT	1 EA
P	TRAUMA FIRST AID MODULE (Refer to Table 3-1 For Components)	1 EA
P	MINOR FIRST AID MODULE (Refer to Table 3-2 For Components)	1 EA
P	QuikClot Combat Gauze™ (Refer to Table 2-1 For Components)	1 EA
	TOURNIQUET, Combat Application/C-A-T®, One-Handed	2 EA



H9003183

Figure 3-2. NSN 6545-01-528-6546, First Aid Kit, Individual (IFAK)

3.3 NSN 6545-01-644-0653, FIRST AID KIT, AFSOC.

Refer to Figure 3-3.

3.3.1 Purpose.**NOTE**

For AFSOC Personnel Use Only!

The AFSOC IFAK provides AFSOC Airmen first aid capability to treat injuries sustained in the line-of-duty.

3.3.2 Installation. When used in various major kits, or standard mobility bags, the USAF activity responsible for assembling the major items will install this kit. When issued as an item of individual equipment on AF Form 538 or 1297, the kit will be worn on a vertical strap of field clothing except during bail out procedures.

3.3.3 Marking. Markings on each kit shall include the nomenclature, NSN (for the kit and case), and the expiration date of the earliest expiring module. Tourniquet (TQ) markings include “TQ” in bold black lettering, NSN, Manufacturer, and date of manufacture/lot.

3.3.4 Inspection. Using activities will return unserviceable kits to the MLF for inspection/reconstitution. Kits are considered unserviceable if the DD form 1574 is not attached or components are expired. Inspect both TQ and pouch. Inspect buckles, rod and locking clips on TQ for chips/cracks, ensure straps are not split, frayed or over stretched. Refer to Paragraph 1.4 and Paragraph 1.5 for further inspection guidance.

3.3.5 Sealing/Tagging. Prior to issuing this kit to using activities, the MLF will ensure the kit has been tagged as outlined in Paragraph 1.3.

3.3.6 Packing. Components are packaged individually. If the integrity of the packaging is compromised, the kit is considered unserviceable and must be returned to the MLF for disposal.

3.3.7 Sustainment. Refer to Paragraph 1.2.2 and Paragraph 1.6.3 for replacement guidance.

Table 3-3. NSN 6545-01-644-0653, First Aid Kit, AFSOC - Components

Notes	Description	Qty/UI
	BAG, USAF JFAK, (Tiger Stripe)	1 ES
	SAFETY HOOK	1 EA
	MARKER, Mini (Sharpie®) 1 EA	1 EA
	EYE SHIELD, Rigid	2 EA
P	AIRWAY, Nasopharyngeal	1 EA
P	CHEST SEAL, Vented (HyFin®)	2 EA
P	PRESSURE BANDAGE	2 EA
P	COMPRESSED GAUZE	2 EA
P	GLOVES, Paper Bound	2 EA
P	CRAVAT	1 EA
P	LUBRICATING JELLY	1 EA
P	DUCT TAPE, Roll	1 RL
P	QuikClot COMBAT GAUZE™	2 EA
	Needle, Cath 6515-01-541-0635	2 EA
	TOURNIQUET, Combat Application/C-A-T®, (One-Handed)	2 EA
	FIELD MEDICAL CARD, DD Form 1380	1 EA



Figure 3-3. NSN 6545-01-644-0653, First Aid Kit, AFSOC

3.4 NSN 6545-01-533-7043, FIRST AID KIT, AIRCRAFT PANEL MOUNTED.

Refer to Figure 3-4. The Aircraft Panel Mounted FAK has been replaced by the Aircraft Panel Mounted JFAK 6545-01-659-8286 and is no longer authorized for procurement. The Aircraft Panel Mounted FAK is however, authorized for use in contingency operations until full implementation of the Aircraft Panel Mounted JFAK.

3.4.1 **Installation.** The using activity responsible for assembling the major items will install kit.

3.4.2 **Marking.** Olive drab external case will be marked with a Red Cross symbol (a 3 x 3-inch solid red cross on a white background). The outer case will include FIRST AID KIT, AIRCRAFT, PANEL MOUNTED in solid black lettering, followed by a black Caduceus and the letters “U” and “S” on either side of the symbol.

3.4.3 **Inspection.** Using activities will return unserviceable kits to the MLF for disposal. Kits are considered unserviceable if the DD form 1574 is not attached, any of the modules or components are expired, or the integrity of the vacuum seal is compromised. Refer to Paragraph 1.4 and Paragraph 1.5 for further inspection guidance. The 36 month inspection benchmark may be adjusted to coincide with aircraft isochronal inspection. Sufficient shelf life must be available on individual components. If this procedure is followed, inspection of the kit can be included in the aircraft inspection checklist and accomplished during routine aircraft downtime.

3.4.4 **Packing.** All modules are in a waterproof/vacuum sealed enclosure. If the integrity of the vacuum seal is open or otherwise compromised, the module is considered unserviceable and must be returned to MLF.

Table 3-4. NSN 6545-01-533-7043, First Aid Kit, Aircraft Panel Mounted - Components

”	Description	Qty/UI
	CASE, First Aid Kit, (Mountable)	1 EA
P	TRAUMA, FIRST AID MODULE (Refer to Table 3-1 FOR COMPONENTS)	1 EA
P	MINOR, FIRST AID MODULE (Refer to Table 3-2 FOR COMPONENTS)	1 EA
	TOURNIQUET, Combat Application/C-A-T®, One-Handed	2 EA
P	QuikClot Combat Gauze™ (Refer to Table 2-1 FOR COMPONENTS)	1 EA
	CPR MODULE (IN LOWER POCKET, Refer to Table 3-4 FOR COMPONENTS)	1 EA



Figure 3-4. NSN 6545-01-533-7043, First Aid Kit, Aircraft Panel Mounted

3.5 NSN 6545-01-659-8286, FIRST AID KIT, AIRCRAFT PANEL MOUNTED JFAK.

Refer to Figure 3-1.

3.5.1 Purpose.**NOTE**

This item is not suitable to wear during bail out procedures.

The Aircraft Panel Mounted JFAK provides Airmen first aid capabilities on board an aircraft or watercraft.

3.5.2 Installation. The using activity responsible for assembling the major items will install kit.

3.5.3 Markings. Olive drab external case will be marked with a Red Cross symbol (3 x 3-inch solid Red Cross on a white background). The outer case will include FIRST AID KIT, AIRCRAFT PANEL MOUNTED in solid black lettering, followed by a black caduceus and the letters "U" and "S" on either side of the symbol.

3.5.4 Inspection. Using activities will return unserviceable kits to the MLF for replacement. Kits are considered unserviceable if the DD form 1574 is not attached or any components are expired. Inspect both TQ and pouch, Inspect buckles, rod, and locking clips on TQ for chips/cracked, ensure straps are not split, frayed, or over stretched. Return unserviceable TQ's to the MLF for disposal. Refer to Paragraph 1.4 and Paragraph 1.5 for further inspection guidance

3.5.5 Sealing/Tagging. Prior to issuing this kit to using activities, the MLF will ensure the kit has been tagged as outlined in Paragraph 1.4

3.5.6 Packing. Components are packaged individually. If the integrity of the packaging is compromised, the component is considered unserviceable and must be returned to the MLF for disposal.

3.5.7 Sustainment. The Aircraft Panel Mounted JFAK has a 60 month non extendable shelf-life. Six months prior to expiration replacements should be ordered, upon expiration the Aircraft JFAK will be turned into the MLF for disposal. Refer to Paragraph 1.2.2 and Paragraph 1.6.3 for replacement guidance.

Table 3-5. NSN 6545-01-659-8286, First Aid Kit, Aircraft Panel Mounted JFAK - Components

Notes	Description	Qty/UI
	CASE, First Aid Kit, (Mountable)	1 EA
P	AIRCRAFT TRAUMA, FIRST AID MODULE	1 EA
P	AIRCRAFT MINOR, FIRST AID MODULE	1 EA
	TOURNIQUET, Combat Application/C-A-T®, One-Handed	2 EA
	CPR MODULE (IN LOWER POCKET, Refer to Table 3-4 FOR COMPONENTS)	1 EA



H2017062A

Figure 3-5. NSN 6545-01-629-8286, First Aid Kit, Aircraft Panel Mounted JFAL




CPR Module H2017066A

Figure 3-6. CPR Module



Aircraft Minor Module H2017066B

Figure 3-7. Aircraft Minor Module

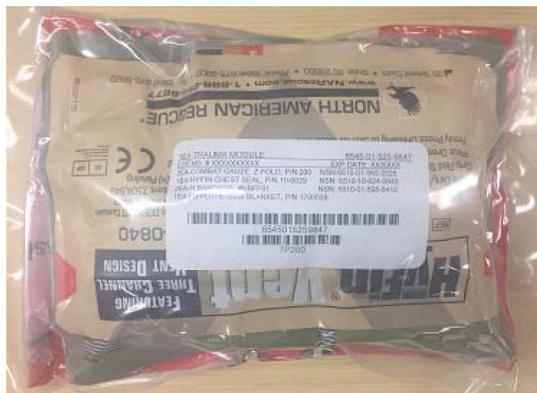

H2017066C

Figure 3-8. Aircraft Trauma Module

3.6 NSN 6545-01-534-0779, FIRST AID KIT, GENERAL PURPOSE.

Refer to Figure 3-9.

3.6.1 Purpose. The General Purpose First Aid Kit provides Airmen first aid capabilities for use in life rafts, airborne lifeboats, vehicle operators/passengers, and to satisfy requirements specified for use in AFOSH Standard 127-50.

3.6.2 Installation. When used in various major kits the USAF activity responsible for assembling the major items will install kit. When issued as an item of individual equipment on AF Form 538 or 1297, the kit will be worn on a vertical strap of field clothing except during bail out procedures.

3.6.3 Marking. Olive drab external case will be marked with a Red Cross symbol (a 3 x 4-inch solid red cross on a white background). The outer case will include GENERAL PURPOSE, FIRST AID KIT in solid black lettering, followed by a black Caduceus with letters “U” and “S” on either side of the symbol.

3.6.4 Inspection. Using activities will return unserviceable kits to the MLF for inspection/reconstitution. Kits are considered unserviceable if the DD form 1574 is not attached, any of the modules or components are expired, or the integrity of the vacuum seal is compromised. Refer to Paragraph 1.4 and Paragraph 1.5 for further inspection guidance.

3.6.5 Sealing/Tagging. Prior to issuing this kit to using activities, the MLF will ensure the kit has been tagged as outlined in Paragraph 1.3.

3.6.6 Packing. All modules are in a waterproof/vacuum sealed enclosure. If the integrity of the vacuum seal is opened or otherwise compromised, the module is considered unserviceable and must be returned to MLF.

3.6.7 Sustainment. Refer to Paragraph 1.2.2 and Paragraph 1.6.3 for replacement guidance

Table 3-6. NSN 6545-01-534-0779, First Aid Kit, General Purpose - Components

Notes	Description	Qty/UI
	CASE, FIRST AID KIT (General Purpose)	1 EA
P	TRAUMA FIRST AID MODULE (Refer to Table 3-1 FOR COMPONENTS)	1 EA
P	MINOR FIRST AID MODULE (Refer to Table 3-2 FOR COMPONENTS)	1 EA
	TOURNIQUET, Combat Application/C-A-T® (One-Handed)	2 EA
P	QuikClot COMBAT GAUZE™ (Refer to Table 2-1 FOR COMPONENTS)	1 EA
P	CPR MODULE (IN LOWER POCKET, Refer to Table 3-4 FOR COMPONENTS)	1 EA



H0901309

Figure 3-9. NSN 6545-01-534-0779, First Aid Kit, General Purpose

3.7 NSN 6545-01-534-0894, SURVIVAL KIT, ESCAPE AND EVASION (E & E).

Refer to Figure 3-10.

3.7.1 Purpose. The Survival Kit, E & E Kit provides first aid items for Airmen to treat injuries sustained in the line-of-duty, aid in operations where military personnel are enabled to emerge from an enemy-held or hostile area.

3.7.2 Installation. When used in various kits, or standard mobility bags, the using activity responsible for assembling the major items will install this kit.

3.7.3 Marking. Markings on the outer module bag label shall be in permanent, legible, and bold upper case lettering and shall include the NSN, module name, lot number and CAGE code. Markings shall also include the kit NSN, DOE, DOM, Lot # and CAGE code in linear bar codes.

3.7.4 Inspection. Using activities will return unserviceable kits to the MLF for inspection/reconstitution. Kits are considered unserviceable if an appropriately marked label and plastic cable is not affixed securing all three modules, any of the modules/components are expired, or integrity of outer bag is compromised. Refer to Paragraph 1.4 and Paragraph 1.5 for further inspection guidance.

3.7.5 Sustainment. Refer to Paragraph 1.2.2 and Paragraph 1.6.3 for replacement guidance.

Table 3-7. NSN 6545-01-534-0894, Survival Kit, Escape and Evasion (E & E) - Components

Notes	Description	Qty/UI
P	SURVIVAL MODULE (Module A)	1 EA
P	MEDICAL MODULE (Module B)	1 EA
	TOURNIQUET, Combat Application/C-A-T® (One-Handed)	1 EA



H0801426

Figure 3-10. NSN 6545-01-537-0894, Survival Kit, Escape and Evasion (E & E)

3.8 6545-01-526-0423, FIRST AID KIT, MASS CASUALTY.

Refer to Figure 3-11.

3.8.1 Purpose. The Mass Casualty Kit provides Airmen first aid capabilities for mass casualty situations.

3.8.1.1 Material requirements consist of one set, First Aid Kit, Mass Casualty and six each litter, rigid poles (6530-01-432-5114) per 100 programmed military personnel.

3.8.1.2 Items pre-positioned under this program will be picked up and maintained by the supporting medical stock record account. DMLSS sites use existing assembly ID "SFAK". The Major Commands (MAJCOMS) will develop a process to ensure material availability to satellite and collocated operating base personnel. This material will be issued to supported units only under predetermined defense conditions or as specified by the MAJCOM. Accordingly, local plans will include procedures for rapid issue of this material upon direction of the major command.

3.8.2 Marking. Each Mass Casualty Kit shall be marked with a red cross, NSN, item identification, shelf life markings, and the quantity and unit of issue (1 EA). Shelf life markings shall also include the inspection/test date, when applicable. Permanent markings on the case shall be as illustrated in Figure 3-11.

3.8.3 Inspection. The MLF is responsible for the overall management and sustainment of these kits under normal circumstances. If deployed, using activities will return unserviceable kits to the MLF for inspection/ reconstitution. Kits are considered unserviceable if a DD Form 1574 is not attached, any of the modules/components in the kit are expired, or integrity of the vacuum seal is compromised. Refer to Paragraph 1.4 and Paragraph 1.5 for further inspection guidance.

3.8.4 Sealing/Tagging. Prior to issuing this kit to using activities, the MLF will ensure the kit has been tagged as outlined in Paragraph 1.3.

3.8.5 Packing. All modules are in a waterproof/vacuum sealed enclosure. If the integrity of the vacuum seal is opened or otherwise compromised, then the module is considered unserviceable and must be returned to MLF.

3.8.6 Sustainment. Refer to Paragraph 1.2.2 and Paragraph 1.6.3 for replacement guidance

Table 3-8. NSN 6530-01-432-5114, LITTER, Raven

LITTER, Raven, (EVACUATION)*NOTE: SUITABLE SUBSTITUTE	6 EA
*Suitable substitute for litter - NSN 6530-01-380-7309	6 EA
STRAP, Webbing, Patient Securing ** ONLY NEEDED FOR SUITABLE SUB	12 EA

Table 3-9. NSN 6545-01-526-0423, First Aid Kit, Mass Casualty - Components

Notes	Description	Qty/UI
	CASE, Mass Casualty	1 EA
P	TRAUMA FIRST AID MODULE (Refer to Table 3-1 For Components)	12 EA
P	MINOR FIRST AID MODULE (Refer to Table 3-2 For Components)	6 EA
P	Quikclot Combat Gauze™ (Refer to Table 2-1 For Components)	12 EA
	TOURNIQUET, Combat Application/C-A-T®, One-Handed	12 EA
	SPLINT MODULE (Refer to Table 3-3 For Components)	8 EA
	CPR MODULE (Refer to Table 3-4 For Components)	6 EA
	BLANKET, Casualty	6 EA



H1501531

Figure 3-11. NSN 6545-01-526-0423, First Aid Kit, Mass Casualty (Front)



H1501532

Figure 3-12. NSN 6545-01-526-0423, First Aid Kit, Mass Casualty (Back)



H1501533

Figure 3-13. NSN 6545-01-526-0423, First Aid Kit, Mass Casualty (Unpacked)



H1501534

Figure 3-14. NSN 6545-01-526-0423, First Aid Kit, Mass Casualty (Packed)

3.9 NSN 6910-01-528-9455, SELF-AID BUDDY CARE (SABC) KIT.

Refer to Figure 3-12.

3.9.1 **Purpose.** The SABC Instructor Kit provides the required Materiel to train Airmen on first aid capabilities.

3.9.2 **Marking.** Each SABC Instructor Kit shall be marked with NSN, CAGE code, nomenclature, and date of manufacture.

3.9.3 **Inspection.** Using activities are responsible for inspecting, repacking, tagging and inventory of the SABC Instructor Kit. The Unit is responsible for funding replacement items through the MLF via appropriate cost center accounts. Kits are considered unserviceable for the next training session if a DD Form 1574 is not attached or any items are missing.

3.9.4 **Sealing/Packing.** Modules contained in the IFAK are in a waterproof/vacuum sealed enclosure. Other components are sealed in a polyethylene bag with slide fastener and excess air removed prior to closing.

3.9.5 **Sustainment.** Using activities requiring replacement modules/components will order them through MLF and funding is the responsibility of requiring activities as stated in Paragraph 1.7. The MLF will order kits, modules, and components through the Medical-Surgical Prime Vendor.

Table 3-10. NSN 6910-01-528-9455, Self-Aid Buddy Care Instructor Kit (SABC) - Components

Description	Qty/UI
CASE, SABC Instructor Kit	1 EA
Individual First Aid Kit (IFAK)	1 EA
AIRWAY, Nasopharyngeal	11 EA

Table 3-10. NSN 6910-01-528-9455, Self-Aid Buddy Care Instructor Kit (SABC) - Components - Continued

Description	Qty/UI
TOURNIQUET, Combat Application/C-A-T®, One-Handed	22 EA
Cotton Bandage Gauzes	11 RO
Emergency Bandage	12 EA
CRAVAT	22 EA
SAM Splint	11 EA
Adhesive Ace Bandage	11 EA
DD Form 1380, U.S. Field Medical Card	1 PD

Table 3-11. Supplemental JFAK SABC Components - P/N RC-0076

Eye Shield	20 EA
Chest Seal	10 EA



H1501535

Figure 3-15. NSN 6910-01-528-9455, Self-Aid Buddy Care (SABC) Instructor Kit



H1501536

Figure 3-16. NSN 6910-01-528-9455, Self-Aid Buddy Care (SABC) Instructor Kit (Packed)

